

H-80 Series Quick Operation Manual

1. Button Functions



Mute Button

Press this button to mute the alert



Knob


Start treatment and adjust device settings



Back Button




Return to the previous operation.


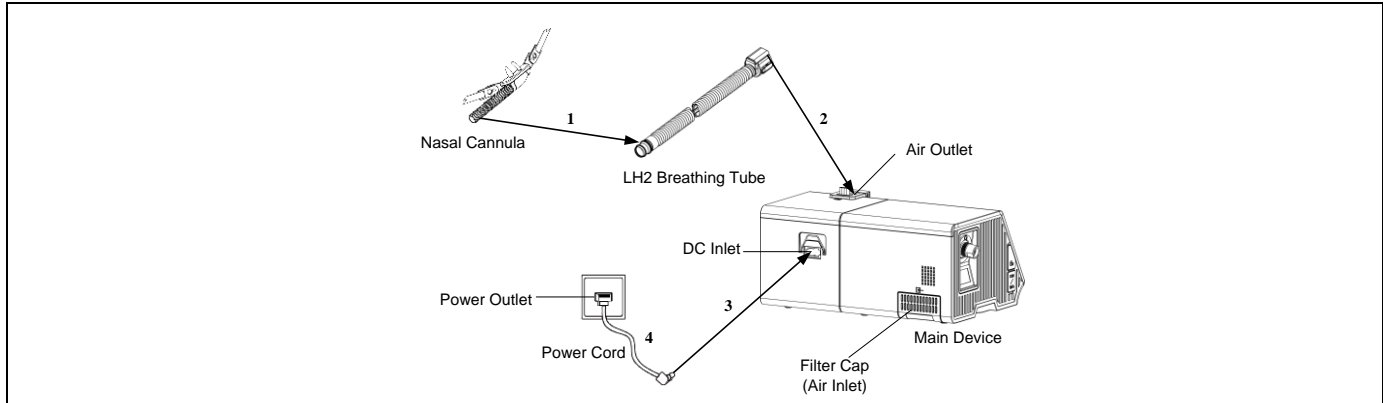
2. Setting up the Device



- A. Connect one end of the LH2 breathing tube to the assembled nasal cannula (as shown in the figure above).
- B. Connect the other end of the LH2 breathing tube to the air outlet of the device firmly (as shown in the figure above).
- C. Connect the power cord to the DC inlet of the main device.
- D. Plug the other end of power cord to the power outlet.

3. Adjusting Device Settings

Connect the power cord and LH2 breathing tube properly, the screen displays the Main Interface shown in the figure left. Turn **the Knob**  to switch among interfaces and options. Press **the Knob**  to conduct parameters setting and confirm options. After finishing adjusting device settings, return to the Main Interface and press **the Knob**  to start treatment.

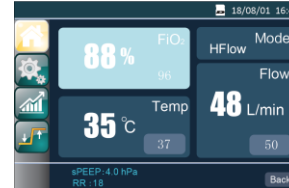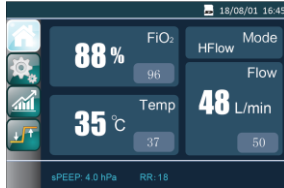

4. Important Note

Before using the device, read all the information in the user manual and memorize the information related to correct operation.