# E-20A-H-O / E-20AJ-H-O Quick Operation Manual


# 1. Button Functions

Mute Button: Press it to mute the alert

Knob: Start treatment and adjust device settings

Ramp Button: Press it to enable the Ramp feature

## 2. Setting up the Device

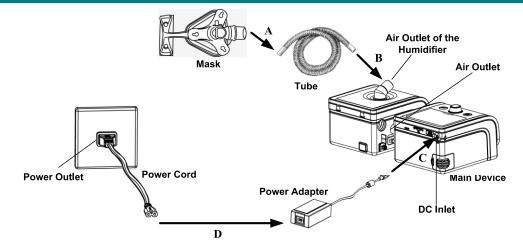

- A. Connect one end of the tube to the assembled mask.
- B. Connect the other end of the tube securely to the air outlet of the main device or of the humidifier, as shown in the figure above.
- C. Connect the power adapter to the DC inlet of the main device.
- D. Connect the power cord to the power adapter.
- E. Plug the power cord into the power outlet.

#### 3. Adjusting Device Settings


A. After the device is connected to power, its screen displays the Main Interface shown in the left figure below (only applies to E-20A-H-O), or the Main Interface shown in the right figure below (only applies to E-20AJ-H-O).





B. From the Main Interface shown above, press and hold **the Ramp Button**  $\triangle$  for three seconds. The screen displays the Initial Setup Interface, as shown in the figure below.



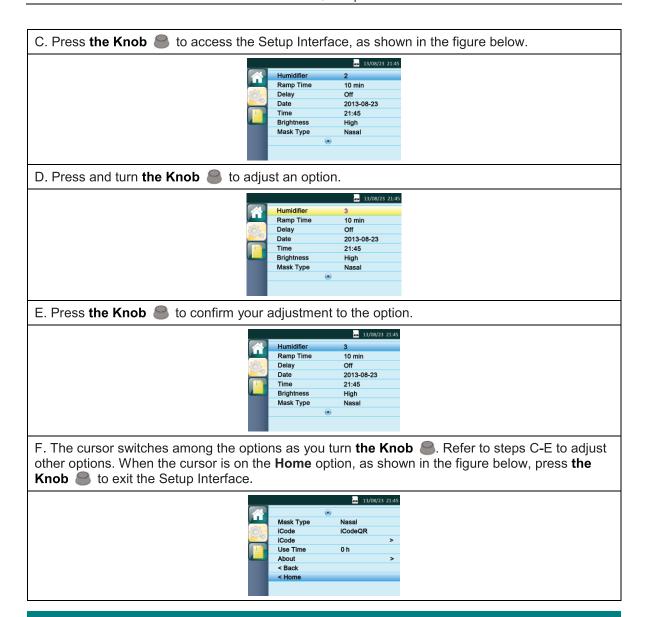

## 4. Starting Treatment

After you finish adjusting device settings and return to the Main Interface, press **the Knob** less to start treatment.

# 5. Important Tips

Please refer to the user manual for any information on the use of the SpO<sub>2</sub> Kit and SD card. Before using the device, please read the entire user manual and memorize all information necessary to ensure proper use of the device.