


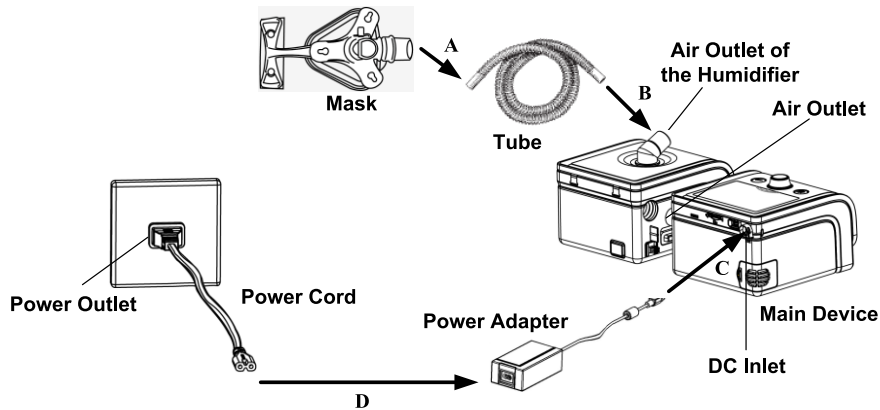


# BPAP System T Series Quick Operation Manual

## 1. Button Functions

-  Mute Button: Press it to mute the alert
-  Knob: Start treatment and adjust device settings
-  Ramp Button: Press it to enable the Ramp feature

## 2. Setting up the Device








- A. Connect one end of the tube to the assembled mask.
- B. Connect the other end of the tube securely to the air outlet of the main device or of the humidifier, as shown in the figure above.
- C. Connect the power adapter to the DC inlet of the main device.
- D. Connect the power cord to the power adapter.
- E. Plug the power cord into the power outlet.


## 3. Adjusting Device Settings

A. After the device is connected to power, its screen displays the Main Interface shown in the first figure below, or the Main Interface shown in the second figure below, or the Main Interface shown in the third figure below, or the Main Interface shown in the fourth figure below.


*Note: The value of blood oxygen saturation (SpO<sub>2</sub>) and pulse rate can only be seen after the SpO<sub>2</sub> Kit is connected to the main device correctly.*

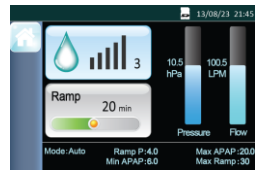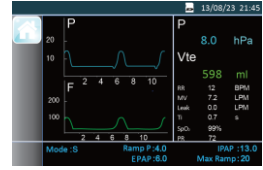 <p>The above interface only applies to T-20S, T-20A.</p>	 <p>The above interface only applies to T-20T.</p>	 <p>The above interface only applies to T-25S and T-25A.</p>	 <p>The above interface only applies to T-25T and T-30T.</p>
--	---	--	---


B. From the Main Interface shown above, press and hold the **Ramp Button**  for three seconds. The screen displays the Initial Setup Interface, as shown in the figure below.




C. Press **the Knob**  to access the Setup Interface, as shown in the figure below.

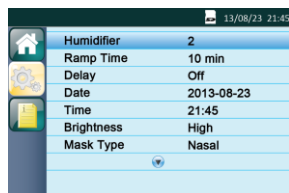

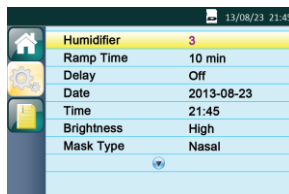


D. Press and turn **the Knob**  to adjust an option.

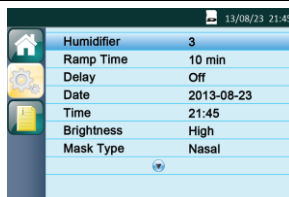


E. Press **the Knob**  to confirm your adjustment to the option.



F. The cursor switches among the options as you turn **the Knob** . Refer to steps C-E to adjust other options. When the cursor is on the **Home** option, as shown in the figure below, press **the Knob**  to exit the Setup Interface.

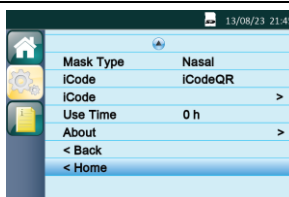

## 4. Starting Treatment

After you finish adjusting device settings and return to the Main Interface, press **the Knob**  to start treatment.

## 5. Important Tips

Please refer to the user manual for any information on the use of the SpO<sub>2</sub> Kit and SD card. Before using the device, please read the entire user manual and memorize all information necessary to ensure proper use of the device.