


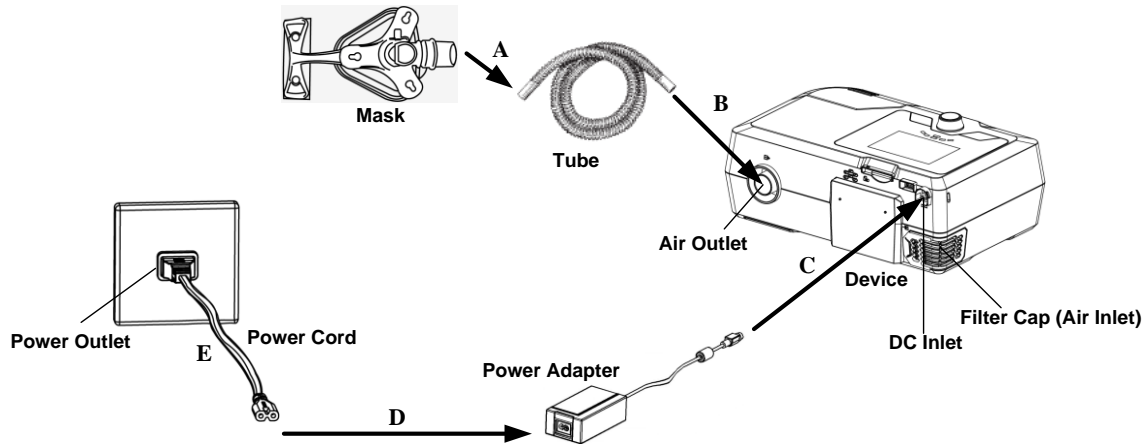


G2S BPAP Series Quick Operation Manual

1. Button Functions

-  Ramp Button: Press it to enable the Ramp feature
-  Mute Button: Press it to mute the alarms
-  Knob: Start treatment and adjust device settings

2. Setting up the Device



- A. Connect one end of the tube to the assembled mask.
- B. Connect the other end of the tube securely to the air outlet of the device, as shown in the figure above.
- C. Connect the power adapter to the DC inlet of the main device.
- D. Connect the power cord to the power adapter.
- E. Plug the power cord into the power outlet.

3. Adjusting Device Settings

A. After the device is connected to power, its screen displays the Main Interface shown in the first figure below, or the Main Interface shown in the second figure below, or the Main Interface shown in the third figure below, or the Main Interface shown in the fourth figure below.


Note: The value of blood oxygen saturation (SpO₂) and pulse rate can only be seen after the SpO₂ Kit is connected to the main device correctly.




The above interface only applies to G2S B20S, G2S B20A.




The above interface only applies to G2S B20T.

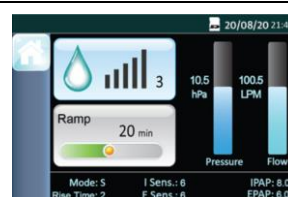


The above interface only applies to G2S B25A, G2S B25S.

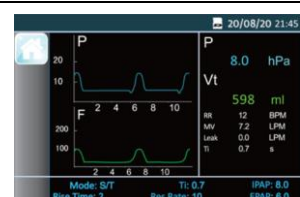


The above interface only applies to G2S B25T, G2S B25VT, G2S B30T, G2S B30VT, G2S B30AT and G2S LAB.


B. From the Main Interface shown above, press and hold **the Ramp Button**  for three seconds. The screen displays the Initial Setup Interface, as shown in the figure below.

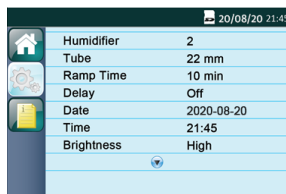


C. Press **the Knob**  to access the Setup Interface, as shown in the figure below.

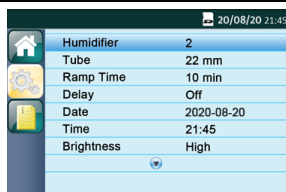

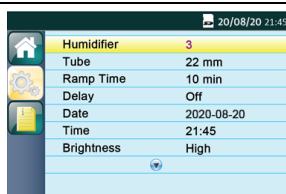


D. Press and turn **the Knob**  to adjust an option.

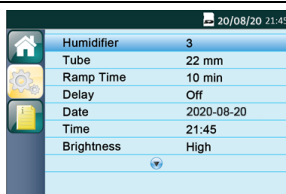


E. Press **the Knob**  to confirm your adjustment to the option.



F. The cursor switches among the options as you turn **the Knob** . Refer to steps C-E to adjust other options. When the cursor is on the **Home** option, as shown in the figure below, press **the Knob**  to exit the Setup Interface.

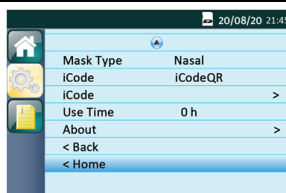

4. Starting Treatment

After you finish adjusting device settings and return to the Main Interface, press **the Knob**  to start treatment.