Mask

Nasal Mask (N5A)

User Manual

Nasal Mask (N5A)

Thank you for choosing BMC's Nasal Mask (N5A). The Nasal Mask is comfortable, flexible and quiet. It is designed to minimize contact with your face, thus ensuring that you feel comfortable during therapy. This user manual provides you with the information you need for the correct use of your mask.

Intended Use

The Nasal Mask (N5A) channels airflow non-invasively to a patient from a positive airway pressure device such as a continuous positive airway pressure (CPAP) or Bi-level system.

The Nasal Mask (N5A) is:

- To be used by adult patients (> 66 lb / 30 kg) for whom positive airway pressure therapy has been prescribed.
- Intended for single-patient re-use in the home environment and multi-patient re-use in the hospital / institutional environment.

CAUTION: In the US, Federal law restricts the sale of this device to sale by or on the order of a physician.

Contraindications

This mask should be used with caution for patients with following conditions:


- Facial deformity.
- Injured facial skin which will contact with the mask during use.

Before Using the Mask

WARNINGS


- The vent holes must be kept clear.
- This mask should only be used with CPAP or bilevel devices recommended by a physician or respiratory therapist.
- The mask should not be used unless the device is turned on. Once the mask is fitted, ensure the device is blowing air.
- Explanation:** CPAP and bilevel devices are intended to be used with special masks (or connectors) which have vent holes to allow continuous flow of air out of the mask. When the device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask vent holes. However, when the device is not operating, insufficient fresh air will be provided through the mask, and the exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation. This applies to most models of CPAP or bilevel devices.
- Not applicable for silicone-allergy user.
- This mask should not be used on patients who are uncooperative, obtunded, unresponsive, or unable to remove the mask.
- Follow all precautions when using supplemental oxygen.
- Oxygen flow must be turned off when the CPAP or bilevel device is not operating, so that unused oxygen does not accumulate within the device enclosure and create a risk of fire.
- Oxygen supports combustion. Oxygen must not be used while smoking or in the presence of an open flame. Only use oxygen in well ventilated rooms.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration varies, depending on the pressure settings, patient breathing pattern, mask, point of application and leak rate.
- The technical specifications of the mask are provided for your clinician to check if it is compatible with the flow generator. If it is used beyond technical specifications or used with incompatible devices, the seal and comfort of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect the function of the flow generator.
- Stop using the Mask and consult your physician or sleep therapist, if you have ANY adverse reaction to the use of the mask.
- Refer to your CPAP or bilevel device manual for details on settings and operational information.
- Remove all packaging before using the mask.
- Images shown here are indicative only. If there is inconsistency between the image and actual product, the actual product shall govern.

Components of the Mask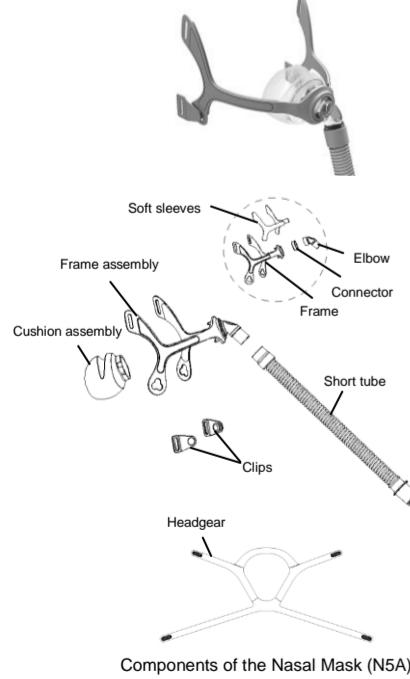


English/Español/Français

Fitting the Mask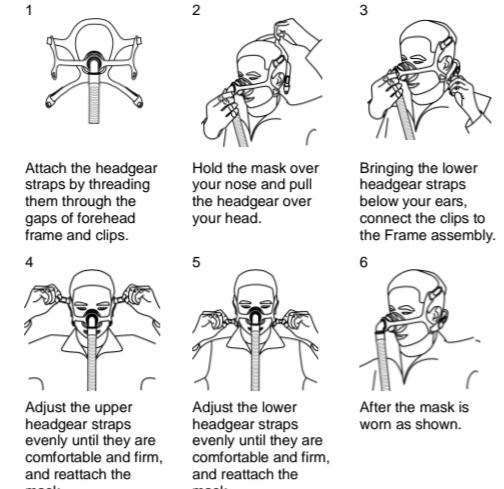

Disassembling the Mask

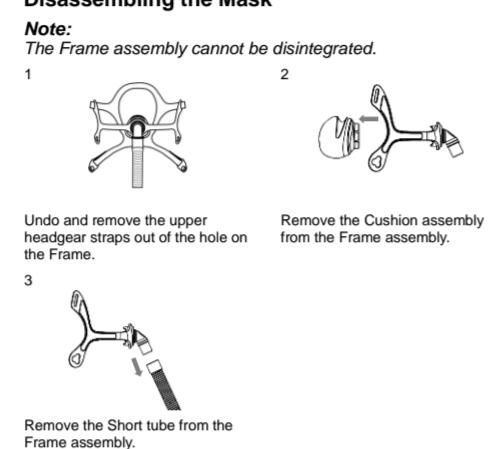Cleaning the Mask at Home

Notes:

- The mask and headgear can only be cleaned through hand washing.
- The headgear may be washed without removing the clips.
- When washing the mask, the Elbow and the Frame cannot be disassembled from the Frame assembly.

CAUTION

- Always follow cleaning instructions and do not use solutions containing bleach, chlorine, alcohol, aromatics, moisturizers, antibacterial agents, or scented oils to clean any part of the system or air tubing, otherwise may damage the mask.
- Exposing any part of the system or tubing to direct sunlight or heat may cause deterioration.
- If any visible deterioration of a component is apparent (cracking, crazing, tears etc), the component should be discarded and replaced.

Daily / After Each Use

- Disassemble the mask components according to the disassembly instructions.
- Thoroughly clean the separated mask components (excluding headgear), by gently rubbing in warm (approx. 30°C) water using mild liquid detergent (e.g., Alconox diluted at 1%) for up to 10 min.
- Use a soft bristle brush to clean the vent.
- Rinse all components well with drinking quality water and allow to air dry out of direct sunlight.
- When all components are dry, reassemble according to the reassembly instructions.


Weekly

Handwash the headgear and all components in warm (approx. 30°C) water using mild liquid detergent (e.g., Alconox diluted at 1%). Rinse the components well and allow them to air dry out of direct sunlight.

Reprocessing the Mask between Patients

This mask should be reprocessed when used between patients. Please refer to the accompanying "Cleaning and Disinfection Instructions" or BMC website www.bmc-medical.com for "Cleaning and Disinfection Instructions". If you do not have internet access, please contact your BMC representative.

Reassembling the Mask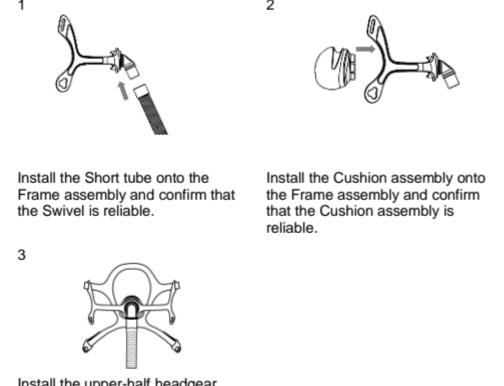


Troubleshooting

Problem	Possible Reason	Possible Solution
Mask won't seal properly or is uncomfortable.	Mask may have been fitted incorrectly or is incorrectly adjusted.	Carefully follow instructions in "Fitting the Mask" section. Make sure the headgear is not over-tightened.
Mask size is wrong.	Mask is not tightened correctly.	Consult your clinician.
Cushion is misaligned or creased on face.	Ensure cushion is properly aligned according to the instructions in "Reassembling the Mask". Make sure cushion has no creases in it.	Tighten both upper and lower headgear straps.
Mask cushion may be dirty.	Clean cushion according to instructions in "Cleaning the Mask at Home".	Reassemble mask.
Mask is too noisy.	Mask is assembled incorrectly.	Reassemble mask.
	Vents are blocked or partially blocked.	Clean the vents according to instructions in "Cleaning the Mask at Home" section.

English/Español/Français

Technical Specifications

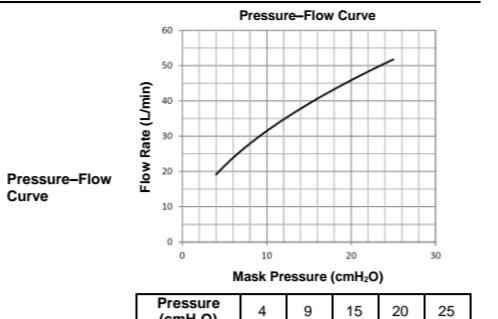Dead Space Information	Dead space is the empty volume of the mask up to the swivel.
Resistance	The dead space of the mask varies according to cushion sizes but is less than 125 mL.

Therapy Pressure	4 to 25 cmH ₂ O.
Resistance	Drop in Pressure measured (average for 4 sizes) at 50 L/min: 0.2 cmH ₂ O at 100 L/min: 0.8 cmH ₂ O

Sound	DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level of the mask is 28 dBA, with uncertainty 3 dBA. The A-weighted sound pressure level of the mask at a distance of 1 m is 20 dBA, with uncertainty 3 dBA.
-------	---

Environmental Conditions	Operating temperature: +5°C to +40°C (41°F to 104°F)
	Operating humidity: 0% ~ 93% relative humidity non-condensing

Storage	Storage and transport: -20°C to +55°C (-4°F to 131°F)
---------	---

Disposal	Dispose of the mask as household refuse.
----------	--

Symbols	The Nasal Mask (N5A) does not contain any hazardous substances and may be disposed of with your normal household refuse.
---------	--

System and Packaging	Caution, consult accompanying documents.
----------------------	--

Notes:	Indicates a Warning or Caution and alerts you to a possible injury or explains special measures for the safe and effective use of the device.
--------	---

Caution	• Deformation facial.
---------	-----------------------

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